

## GoTo®

## LIGHT CURE ORTHODONTIC ADHESIVE

REF GTP (Push syringe) or REF GTT (Tips)

**INDICATIONS FOR USE:** GoTo® is intended for use as a light cure bracket adhesive.

**INSTRUCTIONS FOR USE:**

**Prophylaxis:** Prophylax the enamel surfaces to be bonded with 1st & Final® or plain oil-free pumice. Rinse thoroughly and dry teeth with oil and moisture-free compressed air.

**Etch:** With a cotton pledget or brush, dab the etching agent onto the entire area to be bonded. Allow 30 seconds for etching (60 for deciduous teeth). Avoid placing etching agent on soft tissue. Rinse and dry thoroughly.

**Primer Application:** Apply 1 coat of Assure or Assure Plus and dry with air. If Pro Seal is used, apply a thin coat and light cure the Pro Seal for 10 seconds per tooth. Proceed with application of the paste. Note: A self-etching primer may be substituted for conventional phosphoric acid etchant. Refer to self-etching primer instructions.

**Application of Paste:** Apply adhesive to bracket pad and smear into the bracket mesh. Cure adhesive at close proximity per light manufacturer's recommendation. Once the paste has been properly cured, an active arch wire can be placed immediately.

**Note:** Curing times depend on the intensity of the curing light. The total curing time can range from 6 seconds to 20 seconds. Please refer to the light manufacturer for recommended curing times.

**Bracket Removal:**

To remove GoTo™ use a ligature cutter to grab the adhesive at the bracket/enamel interface and pinch the bracket from the tooth. Remaining material can be removed with the Renew® Finishing System Bur and Renew® Finishing System #383 Point.

**CAUTION:** Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

**i PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

**TEMPERATURE LIMITATION:** Store at room temperature (15-30 Degrees C.) Allow product to reach room temperature prior to use.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**Rx Only:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.



Scan for  
additional  
information



*Reliance Orthodontic Products, Inc.*

Australian Sponsor:  
Emergo Australia  
201 Sussex St. Darling Park  
Tower II, Level 20,  
Sydney, NSW 2000 Australia



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 · [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)