

## Renew<sup>®</sup> Finishing System Points

**INDICATIONS FOR USE:** The Renew<sup>®</sup> Finishing System Points are intended for use in the removal of thin layers of composite paste after debonding and polishing of the surface of enamel, porcelain, metal or composite.

**INSTRUCTION FOR USE:**

1. Ensure that the point is fully seated and gripped in the handpiece collet before use.
2. Maintain handpieces in good working order and correctly lubricated.
3. **IMPORTANT! Do not exceed the maximum speed of 20,000 RPM.** Find the setting for your handpiece that will ensure a speed not to exceed 20,000 RPM and adjust your air pressure accordingly.

**SAFETY IN USE:**

- Eye protection must be worn to protect against ejected particles.
- Surgical mask must be worn to avoid inhalation of aerosol or dust generated.
- Move the point continuously when in use to avoid localized heating.
- Clean and sterilize the points in accordance with the directions below before first use.

**WARNING!!** Used polishing points should be considered as contaminated and appropriate handling precautions should be taken. Gloves, eye protection and a mask should be worn. Other measures may be required if there are specific infection or cross-contamination risks from the patient.

Cleaning /  
Disinfection

Clean and chemically disinfect with a suitable solution (ie Komet DC1) in an ultrasonic device at 40 Degrees C. Rinse thoroughly under running water. Or, place in a suitable rack or bur block, lid open in washer/disinfector to ensure maximum contact with spray jet. Add appropriate cleaning agent per manufacturer recommendations and washer/disinfector mfg instructions. Commence program (pretreat with cold water a minimum of 2 mins., clean with tap water for 5 mins at 55 Degrees C and 0.5% rinsing solution, neutralize for 3 minutes. with demineralized water, rinse for 2 mins. with demineralized water

	and drain. Or, in sterilization bags or in suitable racks/trays, 3 pre-vacuum pulses at 134 Degrees C for 3 minutes with an additional 1 minute of drying time.
Drying	Dry instruments with compressed air or remove from the washer/disinfector and remove any residual moisture with compressed air.
Visual Inspection	Remove any defective devices (chips, cracks...)

**LIMITATIONS OF PROCESSING:** The end of life of the polishing points are determined by wear and damage in use and the points should be inspected for defects during the preliminary cleaning process.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**Rx Only:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**Reorder Information:** Friction Grip (REF# RSPFG), Handpiece (REF# RSPHP), Latch (REF# RSPL)



*Reliance Orthodontic Products, Inc.*

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 · [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)

Australian Sponsor: Emergo  
Australia  
201 Sussex St.  
Darling Park, Tower II, Level  
20, Sydney, NSW 2000  
Australia

Made in the USA



EC

REP

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Scan for additional information

Check out our new learning  
platform Reliance U!  
[www.RelianceU.tech](http://www.RelianceU.tech)