## Please Note

# There is a newer version of this document available at:

RelianceOrthodontics.com/ifu

**Reliance Orthodontic Products, Inc.**  
1540 W. Thorndale Avenue, Itasca, IL 60143 USA  
1-800-323-4348 | Made in USA | Rx Only  
RelianceOrthodontics.com

<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>
<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>
<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>
<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>
<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>
<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>
<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>
<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>
<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>
<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>	<span> </span>

**EMERGO EUROPE**  
Prinssessgracht 20, 2514 AP The Hague, The Netherlands


**AUSTRALIAN SPONSOR EMERGO AUSTRALIA**  
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

## PLASTIC CONDITIONER

### INSTRUCTIONS FOR USE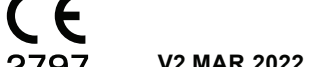

RelianceOrthodontics.com

#### EN - ENGLISH

**INDICATIONS FOR USE:** The Reliance Plastic Appliance Conditioner is intended for use as a conditioner of plastic brackets/acrylic appliances and composite restorations
**INSTRUCTIONS FOR USE:**
**BONDING A PLASTIC BRACKET OR ACRYLIC APPLIANCE:**

With a brush or sponge pellet, apply a thin uniform coat of Reliance Plastic Appliance Conditioner to the plastic surface to be bonded. Wait at least 60 seconds before applying the bonding paste.

**BONDING TO A COMPOSITE RESTORATION:**

Lightly abrade the composite surface with a fine diamond or microetcher, rinse and dry. Apply a thin uniform coat of Reliance Plastic Conditioner and allow to dry for 60 seconds. Apply a thin coat of bonding resin to the tooth. Proceed with bracket placement.

**▲ NOTE: A NEW APPLICATOR IS REQUIRED FOR EACH PATIENT**

⚠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

🇺🇸 **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C). Avoid excessive heat.

⚠ Flammable   ⚠ Harmful to Aquatic Life.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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