## Please Note

# There is a newer version of this document available at:

**RelianceOrthodontics.com/ifu**

**RELANCE ORTHODONTIC PRODUCTS, INC.**

1540 W. Thorndale Avenue,
Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

## RelianceOrthodontics.com

**ITC TRANSLATIONS**

ITC USA

2000 PGA Blvd., Suite 4440
Palm Beach Gardens, FL 33408

**EN - ENGLISH**

**INDICATIONS FOR USE:** The Perfect-A Smile® Pontic Paint is intended for use as a light cure coloring agent for thermoplastic aligners. This patented formula works with clear thermoforming aligner plastics. Contraindication: Perfect A Smile is not recommended for use with cloudy, (C) thermoforming materials.

- To ensure uniformity due to settlement of the product, mix the pontic paint thoroughly with a spatula until uniform. PRODUCT MUST BE MIXED BEFORE EACH USE.
- Remove the amount desired and place on mixing pad. **IMMEDIATELY REPLACE THE LID TO PROTECT LIGHT.**
- Apply 3 coats of the pontic paint with a microbrush. Light cure for 10 seconds after each coat. **NOTE: Shades can be mixed ON A MIXING PAD to achieve different shades.**
- After final coat is applied and light cured, perform a final light cure through the aligner for 20 seconds from the labial and lingual surfaces.

**Please Note: For the most accurate shade matching, fabricate a shade guide by placing 3 coats of each shade to individual tooth surfaces. This shade guide should be fabricated from the same clear aligner material being used.**

⚠️⚠️⚠️ **PRECAUTIONS:** Unpolymerized resins may cause skin irritation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse splash area immediately with copious amounts of water for several minutes. **DO NOT INHALE FUMES. USE IN A WELL VENTILATED AREA.**

⚠️ Dispose of this device/container properly. Marine hazard.

MicroBrush applicators are single use devices. It is required that a new microbrush applicator is used for each application. Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

⚠️ **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C).

Protective eyewear should be worn by the patient and dental staff
**PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

**CONTRAINDICATIONS:** Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

**SAFE DISPOSAL:** Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

**WARRANTY:** Reliance Orthodontic Products Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**CREATING A TEMPORARY TOOTH ON A STONE MODEL**

Roll approximately ½ gram of Perfect A Smile® Pontic Putty between your thumb and index finger.

Place roll of composite on gingival surface of the missing tooth socket and lightly depress ball of composite between thumb and index to produce incisal height slightly shorter than adjacent teeth\*. Precise shaping of the incisal, mesial and distal can be done with an instrument. Light Cure for 30 seconds from 2 (two) angles.

Fractured or worn teeth can be repaired or altered in shape with Perfect A Smile® Pontic Putty Paste.

**\*IMPORTANT!** For a single pontic tooth on the model, create the incisal edge slightly shorter (approx. 0.5mm) than the adjacent teeth. This compensates for the thickness of the aligner material on the facial surface of the adjacent teeth once the aligner socket is painted.

For multiple or adjacent teeth, (for an exact match), form the pontic tooth on the model to the exact incisal height of adjacent teeth. To allow room for the paint on existing teeth in the aligner socket, apply a thin coat of Flow-Tain™ on the facial surfaces of adjacent teeth - similar to the protocol for bleaching relief voids in bleaching trays.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

**REORDER INFORMATION:** #PPA1, #PPA2, #PPA3S, #PPB1, #PPC1, #PPC4, #PPFD3, #PPFL, #PASP, #PASM, #LBOPS

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