Please Note

There is a newer version of this document available at:

RelianceOrthodontics.com/ifu

RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA
2000 PGA Blvd., Suite 4440
Palm Beach Gardens, FL 33408

EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE
EMERGO CONSULTING (UK) LIMITED
c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge
CB24 9BZ
England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE
MedEnvoy Global BV
Leidschendam-Voorburg, Zug Branch Office
Gotthardstrasse 28, 6302 Zug, Switzerland

RESTORE™ DIAMOND POLISHING PASTE

INSTRUCTIONS FOR USE

CE 1639 V3 OCT.2023

RELIANCE®

ORTHODONTIC PRODUCTS

EN - ENGLISH

INDICATIONS FOR USE: Restore™ diamond polishing paste is intended for use in restoring an aesthetic polish on porcelain, zirconia or ceramic surfaces following orthodontic

INSTRUCTIONS FOR USE:

- Rinse the zirconia, ceramic or porcelain surface with water.
- Apply Restore™ liberally to the surface.
- Polish with a Prophy cup or soft buff.
- Rinse with water.

Repeat process above to enhance luster.

TEMPERATURE LIMITATION: Store at room temperature (15-25°C)

CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPS BEFORE EACH PATIENT.

⊕ Avoid contact with skin and eyes.

Protective eyewear should be worn by the patient and dental staff.

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontics Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from use of or suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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