RELIANCE ORTHODONTIC PRODUCTS, INC.
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA
1-800-323-4348 Made in USA | Rx Only
RelianceOrthodontics.com

 RelianceOrthodontics.com

 ITC TRANSLATIONS
ITC USA
2000 PGA Blvd., Suite 4440
Palm Beach Gardens, FL 33408

 EC REP MD

EMERGO EUROPE
Westervoortseindijk 60
6827 AT Arnhem
The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE
EMERGO CONSULTING (UK) LIMITED
c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge
CB24 9BZ
England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE
MedEnvoy Global BV
Leidschendam-Voorburg, Zug Branch Office
Gotthardstrasse 28, 6302 Zug, Switzerland

RELIWAX TEMPORARY MUCOSAL IRRITANT RELIEF

INSTRUCTIONS FOR USE

V0 NOV.2023

 CE
1639

 **RELIANCE®**
ORTHODONTIC
PRODUCTS

SCAN FOR MORE INFORMATION

EN - ENGLISH

INDICATIONS FOR USE: ReliaWax is a flowable, single part, light cure adhesive designed to cover orthodontic appliance protrusions to limit soft tissue irritation, allowing patient to remove as desired.

PERFORMANCE INDICATOR: Securely encompasses orthodontic protrusions and releases upon desired removal.

INSTRUCTIONS FOR USE:

PROPHYLAXIS: A prophylaxis is only necessary if oral debris is present.

APPLICATION: Air-dry desired placement area thoroughly. No primers or further preparation is necessary. Remove syringe cap and install disposable tip. Express syringe to desired location and manipulate with disposable tip and cure with a Dental curing light. Cure for the appropriate duration according to specific light intensity. ReliaWax should fully encompass the orthodontic protrusion and contact the peripheral area of the appliance to achieve a secure bond. Single coat application is ideal, however if more ReliaWax is necessary, an additional layer can be placed as long as the polymerized surface is contaminant free.

REMOVAL: ReliaWax features a malleable cohesive polymerized state allowing ease of removal by dental professional or patients. For dental professionals - utilize a dental instrument to remove the cohesive piece of ReliaWax. For patients - use a toothpick, tweezers or fingers to remove the cohesive piece of ReliaWax.

PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

Crossive

Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

Protective eyewear should be worn by the patient and dental staff.

TEMPERATURE LIMITATION:

Store at room temperature (15-25°C).

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous-waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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