Please Note

There is a newer version of this document available at:

RelianceOrthodontics.com/ifu

Reliance Orthodontic Products, Inc.
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

EC REP **MD**

EMERGO EUROPE
Prinsessgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

RELIAMOLD®
MINI MOLDS

CE

V1 JAN.2022

RELIANCE®

ORTHODONTIC PRODUCTS

INSTRUCTIONS FOR USE

CE

V1 JAN.2022

RELIANCE®

ORTHODONTIC PRODUCTS

CE

V1 JAN.2022

RELIANCE®

ORTHODONTIC PRODUCTS

CE

V1 JAN.2022

RELIANCE®

ORTHODONTIC PRODUCTS

RelianceOrthodontics.com

ENGLISH TRANSLATION
INDICATIONS FOR USE: ReliaMolds are malleable, single-use molds designed to adapt to tooth structure to form various types of composite for functional Orthodontic treatment.

PERFORMANCE INDICATOR: Accurately forms composite as described.

INSTRUCTIONS FOR USE

PROPHYLAXIS: Prophy the enamel surface. Rinse and dry thoroughly.

ETCH: Etch the enamel surface, rinse and dry. **APPLICATION:** Apply a thin coat of Assure Plus and lightly dry (tack light curing at this stage can help reduce mold drift). With an instrument, hold the male end of molds. Fill mold cavity with desired light cure material (FlowTain, LCR, GoTo) and place on tooth surface gently. (Note: if the intention is to bond to a thermoformed aligner, Bond Aligner must be used) Cure for the appropriate duration according to specific light intensity. Gently peel away mold, leaving polymerized adhesive attachment on tooth surface. If necessary, remove any flash with an appliance or rotary instrument.

PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

⚠ Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

Ⓜ Do not reuse.

☀ Keep away from sunlight.

TO REORDER: RMK, RMMP, RMWBL, RMBR3, RMBR5, RMBU, RMC, RMT, BHA

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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