Please Note

There is a newer version of this document available at:

RelianceOrthodontics.com/ifu


Reliance Orthodontic Products, Inc. 1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

EMERGO EUROPE
Prinssegracht 20, 2514 AP The Hague, The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia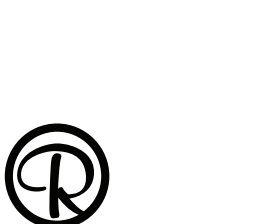

ASSURE® UNIVERSAL BONDING RESIN

INSTRUCTIONS FOR USE

CE 2797 V2 MAR.2022

RELiance® ORTHODONTIC PRODUCTS

RelianceOrthodontics.com

EN - ENGLISH

INDICATIONS FOR USE: Assure® Universal Bonding Resin is intended to increase adhesion to wet or dry, normal and atypical enamel surfaces including: hypocalcified, fluorosed, deciduous enamel and dentin. It will bond to roughened metal (amalgam, gold or stainless steel) and composite restorations without additional primers.

DIRECTIONS FOR USE:

ENAMEL PREPARATION: Prophy the enamel surfaces with 1st & Final® pumice. Rinse and dry thoroughly. Etch the enamel surface for 30 seconds. Rinse and dry thoroughly. If using a self etching scrub the SEP into the enamel for 5 seconds, air dry thoroughly.

APPLICATION OF BONDING RESIN: Apply one coat of Assure® Universal Bonding Resin to each prepared tooth and stroke over to ensure penetration. Lightly air dry. Assure can be light cured to eliminate bracket drift, but not necessary. Proceed with the application of any chemical or light cure paste with bracket.

BONDING TO COMPOSITE, GOLD, AMALGAM, STAINLESS STEEL & PLASTIC: Prophy, rinse and dry. Sandblast, rinse and dry. Apply one coat of Assure® Universal Bonding Resin and lightly dry. Proceed with the application of paste and bracket.

BONDING TO PORCELAIN: Prophy, rinse and dry. Sandblast, rinse and dry. Apply Porcelain Etch, wait 4 minutes. Rinse and dry. Apply one coat of Porcelain Conditioner, wait 2 minutes. Dry for 30 seconds. Apply one coat of Assure, air dry and light cure for 10 seconds. Proceed with bracket placement.

SPECIAL SAFETY NOTICE: Brush applicators are single use devices. It is required that a new brush applicator is used for each application.

⚠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

⚠ Flammable.

REORDER INFORMATION: #AS, #AS2ML

TEMPERATURE LIMITATION: Store at room temperature (15-30 C).

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: Federal law restricts this device to sale or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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