Please Note

There is a newer version of this document available at:

RelianceOrthodontics.com/ifu

RelianceOrthodontics.com

Reliance Orthodontic Products, Inc.
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

EMERGO EUROPE

Prinsessgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

EMERGO EUROPE

Prinsessgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

ASSURE® PLUS

EN - ENGLISH

INDICATIONS FOR USE: Assure® Plus All Surface Bonding Resin is intended to increase adhesion to normal, atypical (wet or dry) enamel surfaces, amalgam, gold, stainless steel, porcelain, zirconia, composite restorations, acrylic temporary/poritic teeth.

DIRECTIONS FOR USE:

ENAMEL PREPARATION

Prophy the surface to be bonded with 1st & Final® oil-free pumice. Rinse and dry. Apply the etching agent to the tooth. Allow 30 seconds for etching. Rinse and dry thoroughly.

NOTE: A self etching primer may also be used in place of phosphoric acid etchant. If using a self etching primer, proceed to application of bonding resin

APPLICATION OF ASSURE® PLUS WITH LIGHT CURE PASTE: Apply one coat of Assure® Plus to all prepared teeth. Lightly dry with air. Proceed with placement of bracket.

APPLICATION OF ASSURE® PLUS WITH CHEMICAL CURE PASTE: Apply one coat of Assure® Plus to all prepared teeth. Lightly dry with air. Light cure for 10 seconds. Proceed with placement of bracket.

BONDING TO COMPOSITE, GOLD, AMALGAM, STAINLESS STEEL OR ACRYLIC TEMPORARIES: Prophy, rinse and dry. Sandblast, rinse and dry. Apply one coat of Assure® Plus All Surface Bonding Resin and lightly dry. Light cure the Assure Plus first for 10 seconds. Proceed with placement of bracket.

BONDING TO PORCELAIN AND ZIRCONIA: Prophy, rinse and dry. Sandblast, rinse and dry. Apply one coat of Porcelain Conditioner and allow to remain on surface for 2 minutes. Thoroughly air dry. Apply one coat of Assure® Plus and lightly dry with air. Light cure for 10 seconds. Proceed with placement of bracket.

REPLACE CAP AFTER USE - AVOID EVAPORATION.

SPECIAL SAFETY NOTICE: Brush applicators are single use devices. It is required that a new brush applicator is used for each application to prevent contamination.

⚠ Flammable

⚠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

⚠ **TEMPERATURE LIMITATION:** Store at room temperature (15–30°C).

WARNING: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: Federal law restricts this device to sale or to the order of a dental professional.

NOTE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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