Please Note

There is a newer version of this document available at:

[RelianceOrthodontics.com/ifu](https://relianceorthodontics.com/ifu)

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue,
Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

[RelianceOrthodontics.com](https://relianceorthodontics.com)

EMERGO EUROPE

Prinssengracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

BOND ALIGNER® CLEAR ALIGNER ADHESIVE

INSTRUCTIONS FOR USE

CE
2797

V2 MAR.2022

RELiance®
ORTHODONTIC
PRODUCTS

[RelianceOrthodontics.com](https://relianceorthodontics.com)

EN - ENGLISH

INDICATIONS FOR USE: Bond Aligner® is intended for use as an orthodontic bonding adhesive for brackets, appliances and button formation to thermoplastic aligner surfaces. It is not indicated for use with cloudy (Class C) thermoforming materials.

INSTRUCTIONS FOR USE:

1. Apply Bond Aligner® clear aligner adhesive to the appliance to be bonded. Aligner should be clean and free of any oral fluids or biofilm where the attachment will be bonded.

2. Seat appliance on thermoplastic aligner and light cure through the aligner for 10-20 seconds (depending on the intensity of curing light).

⚠⚠⚠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

⚠ Corrosive. ⚠ Dispose of properly. Refer to Safety Data Sheet

☞**TEMPERATURE LIMITATION:** Store at room temperature (15-30°C)

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

WARRANTY: Reliance Orthodontic Products Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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