Please Note

There is a newer version of this document available at:

RelianceOrthodontics.com/ifu

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

RelianceOrthodontics.com

EN - ENGLISH

INDICATIONS FOR USE: The Reliance Button Hook Aligner Attachments are intended for use in securing an elastic to a thermoplastic aligner or enamel surface.

INSTRUCTIONS FOR USE:
FOR PLACEMENT OF HOOK ATTACHMENT ONTO A THERMOPLASTIC ALIGNER:

- Aligner must be clean and dry where button hook will be placed. Do not roughen.
- Fill button hook aligner attachment mold with Bond Aligner adhesive and position on the prepared portion of the thermoplastic aligner.
- Light cure for 10 seconds through the Button hook aligner attachment mold and remove mold.
- Light cure the composite button for an additional 10 seconds.
- Secure elastic onto the aligner.

FOR PLACEMENT OF HOOK ATTACHMENT DIRECTLY ONTO ENAMEL SURFACE:

- Prophy, rinse and dry.
- Etch with 37% Phosphoric Acid, rinse and dry.
- Apply one coat of Assure® Plus, lightly dry with air.
- Fill button hook aligner attachment mold with a flowable composite such as Flowtain™ and position attachment on the enamel surface. **Do not use Bond Aligner adhesive when bonding directly to enamel.**
- Light cure through the Button hook aligner attachment mold for 10 seconds and remove mold.
- Light cure the composite button for an additional 10 seconds
- Secure elastic onto the aligner.

WARNING: Reliance Orthodontic Products Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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- Naneste jednu vrstvu Assure © Plus a lehce osušte vzduchem.