## Please Note

# There is a newer version of this document available at:

RelianceOrthodontics.com/ifu

**Reliance Orthodontic Products, Inc.**

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only  
RelianceOrthodontics.com

#### EMERGO EUROPE

Prinsessgracht 20, 2514 AP The Hague, The Netherlands

#### AUSTRALIAN SPONSOR EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

## EXCEL® BONDING ADHESIVE

### INSTRUCTIONS FOR USE

CE 2797

V2 MAR.2022

**RELiance®**  
ORTHODONTIC PRODUCTS

RelianceOrthodontics.com

#### EN - ENGLISH

**INDICATIONS FOR USE:** Excel is intended for use as an orthodontic bonding adhesive for brackets and appliances.

#### **BONDING LARGE ACRYLIC APPLIANCES:**

1. Before beginning the etching process, apply a thin coat of Reliance Plastic Conditioner to the underside of the acrylic appliance, wait 60 seconds.

2. Prophly the involved teeth. Etch the buccal and lingual surfaces of teeth that are covered by the appliance. Do not etch the occlusal surface. Rinse and dry.

3. Condition deciduous enamel with Assure® Plus, air dry.

4. Spatulate Excel® Paste part A&B on a mixing pad. A 1 to 1 ratio produces a 3-minute working time for Excel Regular and 60 seconds for Excel Fast.

5. Apply a generous amount of mixed paste to the appliance.

6. Place the appliance into the mouth and clean up flash.

7. Wait 8 minutes before activating the appliance.

#### **APPLICATION TO BRACKET PAD (INDIRECT BONDING):**

1. Prophly rinse and dry,

2. Etch rinse and dry. Apply 1 coat of Assure® Plus, air dry and light cure.

3. Apply a bead of the mixed paste to the bracket base. Slide the tray into the mouth and hold in place for 3 minutes.

4. After 5 minutes, IDB trays can be removed.

**CAUTION:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.

**TEMPERATURE LIMITATION:** Store at room temperature (15-30°C)

**TO REORDER:** #ERJA, #ERJAF, #ERJB, #TERSA, #TERSAF, #1ERSB

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**RX ONLY:** Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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