Please Note

There is a newer version of this document available at:

[RelianceOrthodontics.com/ifu](https://www.relianceorthodontics.com/ifu)

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

[RelianceOrthodontics.com](https://www.RelianceOrthodontics.com)

EMERGO EUROPE

Prinsessgracht 20, 2514 AP The Hague, The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

O.V.S.™ ORTHODONTIC VENEER SYSTEM

INSTRUCTIONS FOR USE

V2 MAR.2022

[RelianceOrthodontics.com](https://www.RelianceOrthodontics.com)

EN - ENGLISH

INDICATIONS FOR USE: OVS is intended for use as a composite for the fabrication of veneers on undersized teeth.

INSTRUCTIONS FOR USE

VENEER PLACEMENT:

FaciForm™ #12 denotes forms for the PATIENTS RIGHT (REF# FFL)
FaciForm™ #22 denotes forms for the PATIENTS LEFT (REF# FFR)

1. Prophyl, etch, then seal facial and lingual surface of the enamel with Assure® Plus. Air dry.
2. Using a Stick'nPlace™ instrument to hold the FaciForm™ from the facial surface, fill the form with chosen O.V.S.™ composite in a thin, even layer.
3. Place the FaciForm™ with the composite on the prepared tooth. Remove excess from the lingual and gingival margins with a scaler and smooth over the lingual surface.
4. Light cure the facial and lingual surface for 30 seconds.

5. Peel off the FaciForm™ with the incisal margin.
6. Polish and shape the incisal edge and lingual surface as needed per bite (red stripe bur).
7. To place bracket, apply 1 coat of Assure® Plus to tooth, air dry and light cure. Place bracket

⚠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally inhaled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

⚠ **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C).
TO REORDER: #OVS, #OVPA1, #OVPA2, #OVFP1, #FFL, #FFR, #SNP16, #LS3F

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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