Please Note

There is a newer version of this document available at:

RelianceOrthodontics.com/ifu

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

EMERGO EUROPE

Prinssengracht 20, 2514 AP The Hague, The Netherlands

AUSTRALIAN SPONSOR EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

PRO SEAL® & LED PRO SEAL®

PRO SEAL® & LED PRO SEAL®

INSTRUCTIONS FOR USE

RelianceOrthodontics.com

EN - ENGLISH

INDICATIONS FOR USE: PRO SEAL® and LED PRO SEAL® are intended for use as an enamel surface sealant around orthodontic brackets.

INSTRUCTIONS FOR USE:

PROPHYLAXIS: Prophly the enamel surface with 1st & Final® or plain pumice. Rinse and dry thoroughly.

ETCH: Apply etching agent onto the entire area to be bonded. Allow 30 seconds for etching (60 for deciduous teeth). Rinse and dry thoroughly. **NOTE:** Atypical enamel should be first etched, conditioned with one coat of Assure® Plus then lightly dried before the PRO SEAL® is applied.

APPLICATION OF PRO SEAL®: With a bristle brush (do not use a Microbrush), apply a thin, uniform layer to the etched enamel. Stroke over with the same brush to ensure a thin layer and proper coverage. Cure each tooth for 6-10 seconds with a curing light.

IMPORTANT: PRO SEAL® must be cured with a halogen or a broad-wave LED light with high intensity in the 390-440 nm wavelength range. LED PRO SEAL® can be cured with any curing light.

REMOVAL OF SEALANT RESIN: Lightly polish the entire tooth surface with a Renew Finishing #383 rubber point where PRO SEAL® and LED PRO SEAL® has been applied.

⚠️⚠️ CAUTION: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.

Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

⚠️ Corrosivo ⚠️ Dispose of properly. Harmful to aquatic life.

☞☞ TEMPERATURE LIMITATION: Store at room temperature (15-30°C)

WARNING: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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