Please Note

There is a newer version of this document available at:

[RelianceOrthodontics.com/ifu](https://relianceorthodontics.com/ifu)

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue,
Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

[RelianceOrthodontics.com](https://relianceorthodontics.com)

EMERGO EUROPE

Prinsessgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

RENEW® FINISHING SYSTEM POINTS

INSTRUCTIONS FOR USE

CE
2797

V2 MAR.2022

[RelianceOrthodontics.com](https://relianceorthodontics.com)

EN - ENGLISH

INDICATIONS FOR USE: The Renew® Finishing System Points are intended for use in the removal of thin layers of composite and polishing of enamel or artificial substrates.

INSTRUCTION FOR USE:

1. Ensure that the point is fully seated and gripped in the handpiece collet before use.
2. **IMPORTANT! Do not exceed the maximum speed of 20,000 RPM.** Find the setting for your handpiece that will ensure a speed not to exceed 20,000 RPM and adjust your air pressure accordingly.

SAFETY IN USE:

- Eye protection must be worn to protect against ejected particles.
- Surgical mask must be worn to avoid inhalation of aerosol or dust generated.
- Move the point continuously when in use to avoid localized heating.

② Single Use Device

⚠ WARNING!! Used polishing points should be considered as contaminated and appropriate handling precautions should be taken. Gloves, eye protection and a mask should be worn. Other measures may be required if there are specific infection or cross-contamination risks from the patient.

LIMITATIONS OF PROCESSING: The end of life of the polishing points are determined by wear and damage in use and the points should be inspected for defects during the preliminary cleaning process.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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