Please Note

There is a newer version of this document available at:

RelianceOrthodontics.com/ifu

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue,
Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

EMERGO EUROPE

Prinsessgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

RESTORE™ DIAMOND POLISHING PASTE

INSTRUCTIONS FOR USE

V2 MAR.2022

RelianceOrthodontics.com

EN - ENGLISH

INDICATIONS FOR USE: Restore™ diamond polishing paste is intended for use in restoring an aesthetic polish on porcelain, zirconia or ceramic surfaces following orthodontic

INSTRUCTIONS FOR USE:

1. Rinse the zirconia, ceramic or porcelain surface with water.
2. Apply Restore™ liberally to the surface.
3. Polish with a Prophly cup or soft buff.
4. Rinse with water.

Repeat process above to enhance luster.

 **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C)

CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES.

APPLY DISPOSABLE BARRIER SLEEVES/WRAPPS BEFORE EACH PATIENT.

⊕ Avoid contact with skin and eyes.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontics Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from use of or suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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