Please Note

There is a newer version of this document available at:

RelianceOrthodontics.com/ifu

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue,
Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

EMERGE EUROPE

Prinssgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR

EMERGE AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

ULTRA BAND-LOK® & ULTRA BAND-LOK® BLUE

INSTRUCTIONS FOR USE

RelianceOrthodontics.com

EN - ENGLISH

INDICATIONS FOR USE:

Ultra Band-Lok® is intended for use as an orthodontic band cement and for occlusal buildups.

INSTRUCTIONS FOR USE:

Band Cementation:

- Thoroughly prophyl the teeth with 1st & Final®. Rinse, dry and isolate.
- Acid etching of the enamel is not required for cementing bands; however, it is recommended when cementing high stress banded appliances.
- Apply a bead of Ultra Band-Lok® to the inside of the band.
- Seat band, clean off any excess flash. Position light tip in close proximity to the occlusal surface of the tooth. Moving in a slow circle around the top of the band, light cure for 30 seconds. At this point the cement is fully cured, allowing for immediate placement.

If using the Luer-Loc syringes:

- Unscrew and remove the black storage cap from the tip of the syringe.
- Put the tip in place and twist clockwise until the tip is locked in position.
- When the procedure is completed, remove the tip and replace the black storage cap to keep the Ultra Band-Lok® protected from light.

Occlusal Buildup Application: Prophyl, rinse and dry. Etch occlusal surface for 30 seconds, then rinse and isolate. Apply Ultra Band-Lok® in a uniform and shape required to the occlusal surface. Light cure for 20 seconds. Note: a self-etching primer can be utilized in place of phosphoric acid etching.

Important: Ultra Band-Lok® is sensitive to ambient light. Shield exposed unpolymerized product until the product is ready to be light cured.

⚠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse skin area immediately with copious amounts of water for several minutes.

⚠ Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear protective eyewear when using these light curing devices.

⚠ **TEMPERATURE LIMITATION:** Fluoride version store refrigerated (2-8°C). Light Sensitive. Avoid excessive Heat. Allow product to reach room temperature prior to use.

⚠ **NonFluoride version room temperature storage (15-30°C).**

☀ Light Sensitive.

TO REORDER: #JUBLBPP, #UBLPP, #LLUBLBP, #LUBLP, #NFUBLBPP, #NFUBLPP, #NFLULLBPP

WARRANTY: Reliance Orthodontic Products Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: Federal law restricts this device to sale by or under the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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