## Please Note

## There is a newer version of this document available at:

**RelianceOrthodontics.com/ifu**

**RELIANCE ORTHODONTIC PRODUCTS, INC.**

1540 W. Thorndale Avenue, Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

## RelianceOrthodontics.com

**ITC TRANSLATIONS**

ITC USA

2000 PGA Blvd., Suite 4440
Palm Beach Gardens, FL 33408

**EMERGO EUROPE**

Westervoortsewijk 60
6827 AT Arnhem
The Netherlands

**AUSTRALIAN SPONSOR**

**EMERGO AUSTRALIA**
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

**U.K. PERSON RESPONSIBLE**

**EMERGO CONSULTING (UK) LIMITED**

c/o Cr360 – UL International
Compass House, Vision Park
Histon
Cambridge
CB24 9BZ
England, United Kingdom

**SWITZERLAND AUTHORIZED REPRESENTATIVE**

MedEnvoy Global BV
Leidschendam-Voorburg, Zug Branch Office
Gothardstrasse 28, 6302 Zug, Switzerland


## C&C™ AND CHAIN

**US PATENT # 8,403,663**

### INSTRUCTIONS FOR USE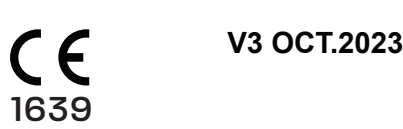

**EN - ENGLISH**

**INDICATIONS FOR USE:** This product is intended for use by a dental professional for extrusion of an impacted tooth.

**INSTRUCTIONS FOR USE:**

- Prophy the enamel surface. Rinse and dry. Acid etch the enamel for 30 seconds, Rinse and dry.
- Apply one coat of Assure® Plus. Lightly air dry.
- Fill the mold to the top edge with paste ensuring the chain is covered. Do not overfill form.
- Quickly place the loaded form on the conditioned, exposed tooth surface.
- Light cure per manufacturer instructions. Gently remove mold from polymerized composite.
- Surgically close the site, as necessary, attaching the gold chain to the appropriate tooth or archwire.

⊕ **Note: C&C Applicators and Self mixing tips are single use devices. A new applicator mold/chain and tip is needed for each patient.**

⊕ Irritant. Bulk Gold Chain is Non-Hazardous and Nickel-Free.

Protective eyewear should be worn by the patient and dental staff

**PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

**CONTRAINDICATIONS:** Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

**SAFE DISPOSAL:** Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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