Please Note

There is a newer version of this document available at:

RelianceOrthodontics.com/ifu

RELIANCE

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue,
Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

EMERGO EUROPE

Prinsessgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

KOBAYASHI TIE HOOKS

INSTRUCTIONS FOR USE

CE
2797

V3 JUN.2022

RelianceOrthodontics.com

EN ENGLISH TRANSLATION

INDICATIONS FOR USE: Reliance Kobayashi Tie Hooks are intended for use in ligation of archwires and hooks to orthodontic brackets and elastics.

INDICATIONS FOR USE

- 1.Remove single Kobayashi Tie Hook from container. Perform hot or cold sterilization.
- 2.Position open end of the tie hook over the bracket tie wing.
- 3.Twist the two ends (approximately 4 times) until it is firmly seated against the bracket base.
- 4.Cut excess wire off close to the base without compromising the twist.
- 5.Bend the loop to the desired angulation.
- 6.Attach elastic, if desired.

② Single Use Device

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

⚠️⚠️⚠️ **WARNING:** This product can expose you to Nickel (metallic), which is known to the State of California to cause cancer. For more information, go to www.P65Warnings.ca.gov.

⚠️ **Flammable as dust** ⚠️ **Dispose of Properly**

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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