

**INDICATIONS FOR USE:** Orthodontic Prophylaxis Pumice for use in cleaning the tooth surface prior to acid etching.

**PRODUCT INFORMATION:**

Pre-packaged and pre-mixed orthodontic prophylaxis pumice.

No flavoring, fluorides, oils or chemicals which could leave a film that may inhibit or weaken the bond.

1st & Final<sup>®</sup> prophylaxis pumice should be used after removal of brackets, bands and bonding material as a final clean up for that bright, new smile!

**INSTRUCTIONS FOR USE**

1. Remove foil barrier from 1st & Final<sup>®</sup> prophylaxis cup.
2. Using a prophylaxis angle with prophylaxis cup attached (example, Reliance Prep Step<sup>™</sup> Disposable Prophylaxis Angle), fill the prophylaxis cup with paste and establish a firm finger rest.
3. Bring the prophylaxis angle cup almost in contact with the tooth surface before operating the prophylaxis angle.
4. Replenish the supply of paste in the prophylaxis cup, as needed. Do not overfill.
5. After each patient, dispose of prophylaxis angle and unused prophylaxis cup of 1st & Final<sup>®</sup>. Do Not Re-use.

NOTE: Each individual unit of 1st & Final<sup>®</sup> contains sufficient paste for one full prophylaxis procedure.

**IMPORTANT:** Excessive prophylaxis paste should be expectorated, not swallowed.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.