



ACCESS™
Cheek Retractor

V8.1 1/15

Ref# CH1 - Adult Cheek Retractor

Lot #

Ref# CH2 - Pedo Cheek Retractor

INDICATIONS FOR USE: The **ACCESS™** Cheek Retractors are intended for use in pulling the cheeks back from the buccal and labial surfaces of the teeth during orthodontic procedures

PRODUCT FEATURES: Engineered for autoclave, dry heat or chemical sterilization.

IMPORTANT!! When sterilized using a dry heat sterilizer, such as a Cox or Dentronix unit, DO NOT PLACE ANY OTHER INSTRUMENTS ON TOP OF THE **ACCESS™** CHEEK RETRACTOR OR CHEEK RETRACTOR PARTS! Load other instruments in the tray FIRST, then place the **ACCESS™** CHEEK RETRACTOR ON TOP of the other instruments. When heat sterilized following these guidelines, the **ACCESS™** Cheek Retractor will provide you with an optimum number of cycles up to 350° Fahrenheit.

INSTRUCTIONS FOR USE:

To Adjust Tension: The **ACCESS™** Cheek Retractor comes with tension adjusted for most situations. A "straight" bow is normal and should be adequate for majority of patients when inserted into the mouth.

To Reduce Tension: Remove the cheek retractors from the bow and gently bend the stainless steel bow to bring the two plastic ends of the bow closer together. This may have to be repeated to hold the

adjustment to get a “continuous curve” to the stainless steel bow. The opposite will increase tension. The cheek retractor is then ready to insert into the mouth.

CLEANING NOTE:

If after several uses and sterilization cycles you observe any discoloration on the retractor, use “SCOTCH BRITE® – Never Scratch™ Wool Soap Pads” to gently and quickly remove the spots. These pads are readily available at any supermarket.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 · www.Relianceorthodontics.com



Scan Barcode for additional information

Australian Sponsor: Emergo Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia



EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands