



Assure®

PLUS

V13.2.20

ALL SURFACE HYDROPHILIC BONDING RESIN FOR WET OR DRY ENVIRONMENT

INDICATIONS FOR USE: Assure® Plus All Surface Bonding Resin is intended to increase adhesion to normal, atypical (wet or dry) enamel surfaces, amalgam, gold, stainless steel, porcelain, zirconia, composite restorations, acrylic temporary/pontic teeth.

DIRECTIONS FOR USE:

ENAMEL PREPARATION

Prophy the surface to be bonded with 1st & Final® oil-free pumice. Rinse and dry. Apply the etching agent to the tooth. Allow 30 seconds for etching. Rinse and dry thoroughly.

NOTE: A self etching primer may also be used in place of phosphoric acid etchant. If using a self etching primer, proceed to application of bonding resin

Application of Assure® Plus with Light Cure Paste: Apply one coat of Assure® Plus to all prepared teeth. Lightly dry with air. Proceed with placement of bracket.

Application of Assure® Plus with Chemical Cure Paste: Apply one coat of Assure® Plus to all prepared teeth. Lightly dry with air. Light cure for 10 seconds. Proceed with placement of bracket.

BONDING TO COMPOSITE, GOLD, AMALGAM, STAINLES STEEL OR ACRYLIC TEMPORARIES: Prophy, rinse and dry. Sandblast, rinse and dry. Apply one coat of Assure® Plus All Surface Bonding Resin and lightly dry. Light cure the Assure Plus first for 10 seconds. Proceed with placement of bracket.

BONDING TO PORCELAIN AND ZIRCONIA: Prophy, rinse and dry. Sandblast, rinse and dry. Apply one coat of Porcelain Conditioner and allow to remain on surface for 2 minutes. Thoroughly air dry. Apply one coat of Assure® Plus and lightly dry with air. Light cure for 10 seconds. Proceed with placement of bracket.

Replace cap after use - Avoid evaporation.

Special Safety Notice: Brush applicators are single use devices. It is required that a new brush applicator is used for each application to prevent contamination.



Flammable



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

To Reorder: REF# PLUS (Assure® Plus, 6mL), REF# UAP50 (Unidose Assure Plus, 50 x 0.20ml Capsules), REF# UAP100 (Unidose Assure Plus, 100 x 0.20ml Capsules) REF# PLUS2ML (Assure® Plus, 2mL)



TEMPERATURE LIMITATION: Store at room temperature (15-30°C).

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: Federal law restricts this device to sale or on the order of a dental professional.

Made in the USA



Reliance Orthodontic Products, Inc.



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com

Scan for more information



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Australian Sponsor:
Emergo Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia

MD

Check out our new learning
platform Reliance U!
www.RelianceU.tech