

BLUE HIGH VISCOSITY GEL ETCHANT with BAC
Contains Benzalkonium Chloride (BAC), which may cause mucosal
irritation

35% Phosphoric Acid

REF BEGS

INDICATIONS FOR USE: Reliance Blue High Viscosity Gel Etchant with BAC is intended for use as an etching agent of dentin or enamel to prepare the surface for bonding.

PRODUCT FEATURES

- Contains BAC (benzalkonium chloride), an antimicrobial that has been shown to inhibit bacteria up to 7mm when challenged (in-vitro) with actinomyces viscosus.* Note: The inclusion of benzalkonium chloride has not been shown to correlate with a reduction in secondary decay in patients. In-vivo clinical studies to evaluate the effects of benzalkonium chloride on oral bacteria or caries have not been performed.
- Highly visible blue shade.

INSTRUCTIONS FOR USE

1. Unscrew and remove the storage cap from the tip of the syringe.
2. Snap the needle tip dispenser in place and twist clockwise until needle is locked.
3. Isolate the teeth for etching. With a cotton pledget or brush, dab the etching agent onto the entire area to be bonded. Do not rub the etching agent onto the enamel. Allow 30 seconds for etching.



CAUTION: Avoid placing etching agent on soft tissue.

4. Rinse thoroughly (at least 10 seconds per tooth) to completely remove all traces of etchant.
5. When procedure is completed, remove needle tip and replace the white storage cap to keep the etchant clean and moist.



CAUTION: DO NOT REUSE TIPS. A NEW TIP IS REQUIRED FOR EACH PATIENT TO AVOID CONTAMINATION RISK.

*Chan, D.C.N., Lo, W.W. Residual Antimicrobial Action of Benzalkonium Chloride Containing Etchant. J. Dent Res 1994; 73(IADR Abstract): P226; Abstr No. 995



CAUTION: Corrosive. Etching liquids or gels (phosphoric acid) are severe eye and tissue irritants. Injury may result if the etchant is allowed to remain on the skin or mucosa for extended periods of time. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Refer to the MSDS for further information.



TEMPERATURE LIMITATION: Store at room temperature (15-30°C.)



Protect from light.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



Scan for more information



Reliance Orthodontic Products, Inc.



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 · www.RelianceOrthodontics.com

Australian Sponsor: Emergo
Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Check out our new
learning platform
Reliance U!
www.RelianceU.tech

Made in the USA