

Reliance Bonding Resins

WITH FLUORIDE

BONDING SYSTEM FOR DIRECT AND INDIRECT BONDING

INDICATIONS FOR USE: Reliance Bonding Resins are intended for use in direct and indirect bonding of orthodontic brackets and appliances.

INSTRUCTIONS FOR USE:

1. Prepare the tooth surface (Prophy, etch, rinse, dry)
2. Mixing: Dispense equal amounts of Bonding Resin Part A&B onto a mixing pad. Using a brush or sponge pellet, mix the two resins thoroughly (5-10 seconds).
3. Application: Apply a thin coat of mixture onto the etched surface. The working time of the Bonding Resin is 40 seconds when measured from the start of mixing. It is not necessary to wait for the bonding resin to set before proceeding to the application of the adhesive.

Special Safety Notice: Brush applicators are single use devices. It is required that a new brush applicator is used for each application.



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.



TEMPERATURE LIMITATION: Store at room temperature (15-30 °C)

TO REORDER: REF# S7AF – 7 gm Bonding Resin A w/Fluoride, REF# S7B – 7gm Bonding Resin B, REF# S20AF – 20 gm Bonding Resin A w/Fluoride, REF# S20B – 20 gm Bonding Resin B

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com

Check out our new learning
platform Reliance U!
www.RelianceU.tech

EC

REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



Scan for more information

Made in the USA

Australian Sponsor:
Emergo Australia
201 Sussex St. Darling
Park
Tower II, Level 20,
Sydney, NSW 2000
Australia