

Enhance™ L.C.

LIGHT CURE ADHESION BOOSTER

(Ref# LCE6)

INDICATIONS FOR USE: Enhance™ L.C. is intended for use as a light cure adhesion booster to: fluorosed, hypocalcified or deciduous enamel, saliva contaminated enamel, enamel where maximum adhesion is required, metal surfaces (gold, amalgam, stainless steel) and acrylic, porcelain or composite restorations.

INSTRUCTIONS FOR USE:

1. Prophylax and rinse thoroughly.
2. Etch enamel for 30 seconds. Rinse thoroughly. Lightly air dry.
3. Dispense Enhance™ L.C. into a mixing well. With a brush apply 3-4 generous coats to the enamel surface. After the last coat, lightly dry the surface with air. The enamel will now appear shiny.
4. Apply a thin coat of Light Bond™ Sealant, PRO SEAL®, L.E.D. PRO SEAL® or any light cure sealant onto the primed enamel. Light cure for 10 seconds (PRO SEAL®/L.E.D. PRO SEAL® - 20 seconds).
5. Apply your bracket with bonding paste and light cure as usual. When bonding to a composite or acrylic surface, lightly roughen the surface with a fine diamond, then follow steps 3-5. When bonding to a metal surface, roughen with a medium diamond bur or sandblast, then follow steps 3-5.

For porcelain surfaces, condition first with Assure® All Surface Bonding Resin and Reliance Porcelain Conditioner.

CAUTION: Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

Special Safety Notice: Brush applicators are single use devices. It is required that a new brush applicator is used for each application.



Flammable



TEMPERATURE LIMITATION: Store at room temperature (15-30 Degrees C).

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com

Check out our new learning platform Reliance U!
www.RelianceU.tech



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



Australian Sponsor:
Emergo Australia
201 Sussex St.
Darling Park
Tower II, Level 20,
Sydney, NSW 2000
Australia

Made in the USA

Scan for more information