

Reliance

GEL ETCHANT IN SYRINGE

INDICATIONS FOR USE: Reliance Gel Etchant is 37% phosphoric acid (H₃PO₄) polymer-thickened and intended for use on enamel to prepare the surface for bonding.

PRODUCT FEATURES

Reliance Gel Etchant will effectively etch and demineralize enamel producing the necessary retentive surface for successful bonding. Our specially formulated Gel Etchant leaves no silica residue.

INSTRUCTIONS FOR USE

1. Unscrew and remove the white storage cap from the tip of the syringe.
2. Snap the needle tip dispenser in place and twist clockwise until needle is locked.
3. Isolate the teeth for etching. With a cotton pledget or brush, dab the etching agent onto the entire area to be bonded. Do not rub the etching agent onto the enamel. Allow 30 seconds for etching.

CAUTION: Avoid placing etching agent on soft tissue.

4. Rinse thoroughly (at least 10 seconds per tooth) to completely remove all traces of etchant.
5. When procedure is completed, remove needle tip, discard need tip and replace the white storage cap to keep the etchant clean and moist.



CAUTION: Etching gels (contain phosphoric acid) are severe eye and tissue irritants. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Injury may result if etchant is allowed to remain on the skin or mucosa for extended periods of time.

CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPPS BEFORE EACH PATIENT. TIPS ARE SINGLE USE DEVICES. A NEW TIP IS REQUIRED FOR EACH PATIENT.



TEMPERATURE LIMITATION: Store at room temperature (15-30 °C)

TO REORDER: REF# EGSS- 6.5g Gel Etchant in Syringes w/Tips,
REF#EGTIPS-100 Gel Etchant Syringe Tips (Tips only)

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com



Scan for more information

Check out our new learning platform Reliance U!
www.RelianceU.tech

EC

REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Australian Sponsor:
Emergo Australia
201 Sussex St.
Darling Park
Tower II, Level 20,
Sydney, NSW 2000
Australia

Made in the USA