

GoTo[®]
FLUORESCING
LIGHT CURE ORTHODONTIC ADHESIVE
REF GTP (Push syringe) or REF GTT (Tips)

INDICATIONS FOR USE: GoTo[®] is intended for use as a light cure bracket adhesive.

Performance Indicator: Bonds to enamel, porcelain, composite and metal for duration of treatment.

INSTRUCTIONS FOR USE:

Tooth Preparation: Prophyl the enamel surfaces with 1st & Final[®], rinse and dry thoroughly.

Etching: Apply etching agent onto the entire area to be bonded. Allow 30 seconds for etching. Rinse and dry thoroughly.


Primer Application: Apply 1 coat of Assure Plus and dry with air.


Application of Paste to bracket: Apply adhesive to bracket pad and smear into the bracket mesh. Apply bracket to the primed tooth surface. Light cure adhesive at close proximity for 10 seconds.

Note: GoTo will fluoresce under the presence of a blacklight.

Bracket and Adhesive Removal:

To remove GoTo[™] use a ligature cutter to grab the adhesive at the bracket/enamel interface and pinch the bracket from the tooth. Remaining material can be removed with a Renew[®] Finishing System Bur and Renew[®] Finishing System #383 Point.

PRECAUTIONS:  Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Avoid placement of etching agent on soft tissue.

 Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPS BEFORE EACH PATIENT.



TEMPERATURE LIMITATION: Store at room temperature (15-30 Degrees C.)

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



Scan for
additional
information



RELIANCE

ORTHODONTIC
PRODUCTS



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1- 800- 323- 4348 • www.RelianceOrthodontics.com

Made in
the USA



Australian Sponsor:
Emergo Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia

Check out our new learning
platform Reliance U!
www.RelianceU.tech

EC

REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

NOTICE: Report any serious incident to

Reliance and the competent authority of the state in which the user and/or patient is established.