

Page 1 of 1	Document # FORM 4.3.3	Version # 1.1	Revision Date 12/15/2017	Form
Approved by P. Ganga	Written by P. Wendland	Document Owner S. Ganga	Document Title Labeling Proof Form	

Purpose: Use this form to track the current master proof for all Reliance Orthodontic Products Inc. labeling material including use instructions, device labels, SDS, etc.

Instructions: Step 1: Identify labeling identifier information

Label Identifier	Version #	Revision Date
Label Title		

Step 2: Attach affix, print or imprint the label to this form. Use full-sized master label.

Use this page, the backside, and additional pages, as necessary. When multiple pages or sheets are used, secure and paginate.

Step 3: Identify the changes from previous to current version of labeling

Changes (or new edition)

V4 3/18



0086

PREP STEP™
Disposable Prophyl Angles

4 WEB • SOFT • 144 INDIVIDUALLY WRAPPED ANGLES

REF DPA

INDICATIONS FOR USE: Disposable prophyl angles are intended for use in cleaning enamel and non-enamel surfaces.

INSTRUCTIONS FOR USE: To ensure that the Disposable Prophyl Angle run smoothly, it is recommended to run them for 5 seconds before operation. Place angle on nose cone of dotrix style handpiece and proceed with use. The prophyl cup turns in a counter-clockwise direction with the handpiece set in the "forward" direction. Use "reverse" to obtain clockwise rotation.



CAUTION: Do not operate at speeds higher than 3000 RPM. **DO NOT REUSE.**

Please remember to dispose of used angles properly.

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Ave., Itasca, Illinois 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com

1-800-323-4348

Rx Only. U.S. Federal law restricts this device to sale by or on the order of a dental professional. MADE IN U.S.A

EC REP

EMERGO EUROPE
Prinsessgracht 20,
2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA
Level 20, Tower II,
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia

ES or NO or NIA

Reviewer: _____

Reviewer: _____

Reviewer: _____

Reviewer: _____

Reviewer: _____

Reviewer: _____

Reviewer: _____

Reviewer: _____

Label Approved By/Date: _____

Website Update: _____

Copies Provided to Customer Service: _____

Label Issue on Date: _____

Label Obsolete on Date: _____