

**INDICATIONS FOR USE:**

Ultra Band-Lok® is intended for use as an orthodontic band cement and for occlusal buildups.

**Performance Indicator:** Bonds to enamel, porcelain, composite and metal for duration of treatment.

**INSTRUCTIONS FOR USE:**

**Band Cementation:**

- a) Thoroughly prophylax the teeth with 1st & Final®. Rinse, dry and isolate.
- b) Acid etching of the enamel is not required for cementing bands; however, it is recommended when cementing high stress banded appliances
- c) Apply a bead of Ultra Band-Lok® to the inside of the band.
- d) Seat band, clean off any excess flash. Position light tip in close proximity to the occlusal surface of the tooth. Moving in a slow circle around the top of the band, light cure for 30 seconds. At this point the cement is fully cured, allowing for immediate placement.

If using the Luer-Loc syringes:

- a) Unscrew and remove the black storage cap from the tip of the syringe.
- b) Put the tip in place and twist clockwise until the tip is locked in position.
- c) When the procedure is completed, remove the tip and replace the black storage cap to keep the Ultra Band-Lok® protected from light.

**Occlusal Buildup Application:** Prophylax, rinse and dry. Etch occlusal surface for 30 seconds, then rinse, dry and isolate. Apply Ultra Band in the amount and shape required to the occlusal surface. Light cure for 20 seconds. Note: a self-etching primer can be utilized in place of phosphoric acid etching.

**Important:** Ultra Band-Lok® is sensitive to ambient light. Shield exposed unpolymerized product until the product is ready to be light cured.



**PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.



Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.



**TEMPERATURE LIMITATION:**

Fluoride version store refrigerated (2-8°C). Light Sensitive. Avoid excessive Heat. Allow product to reach room temperature prior to use.



NonFluoride version room temperature storage (15-30°C).



Light Sensitive.

**TO REORDER:**

Syringes with Fluoride: Ref #UBLBPP for Ultra Band-Lok® Blue push syringe

Ref# UBLPP for Ultra Band-Lok® Natural push syringe

Ref# LLUBLBP for Ultra Band-Lok® Blue Shade Luer-Loc syringe with 10 tips

Ref# LLUBLP for Ultra Band-Lok® Natural Shade Luer-Loc syringe with 10 tips

Syringes Non-Fluoride: Ref #NFUBLBPP for Ultra Band-Lok® Blue push syringe NonFluoride

Ref# NFUBLPP for Ultra Band-Lok® Natural push syringe NonFluoride

Ref# NFLLUBLBP for Ultra Band-Lok® Blue Shade Luer-Loc syringe NonFluoride with 10 tips

**WARRANTY:** Reliance Orthodontic Products Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**Rx Only:** Federal law restricts this device to sale by or on the order of a dental professional.



**RELIANCE®**

ORTHODONTIC  
PRODUCTS



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.

1- 800- 323- 4348 • [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Australian Sponsor: Emergo Australia  
201 Sussex St. Darling Park  
Tower II, Level 20,  
Sydney, NSW 2000 Australia

Made in the USA

Check out our new learning platform Reliance U!  
[www.RelianceU.tech](http://www.RelianceU.tech)



Scan for more information



NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.